



labXpert

Your Expert in Lab for Data Analysis


mindray
healthcare within reach




labXpert

Your Expert in Lab for Data Analysis

New working procedure with labXpert brings efficiency

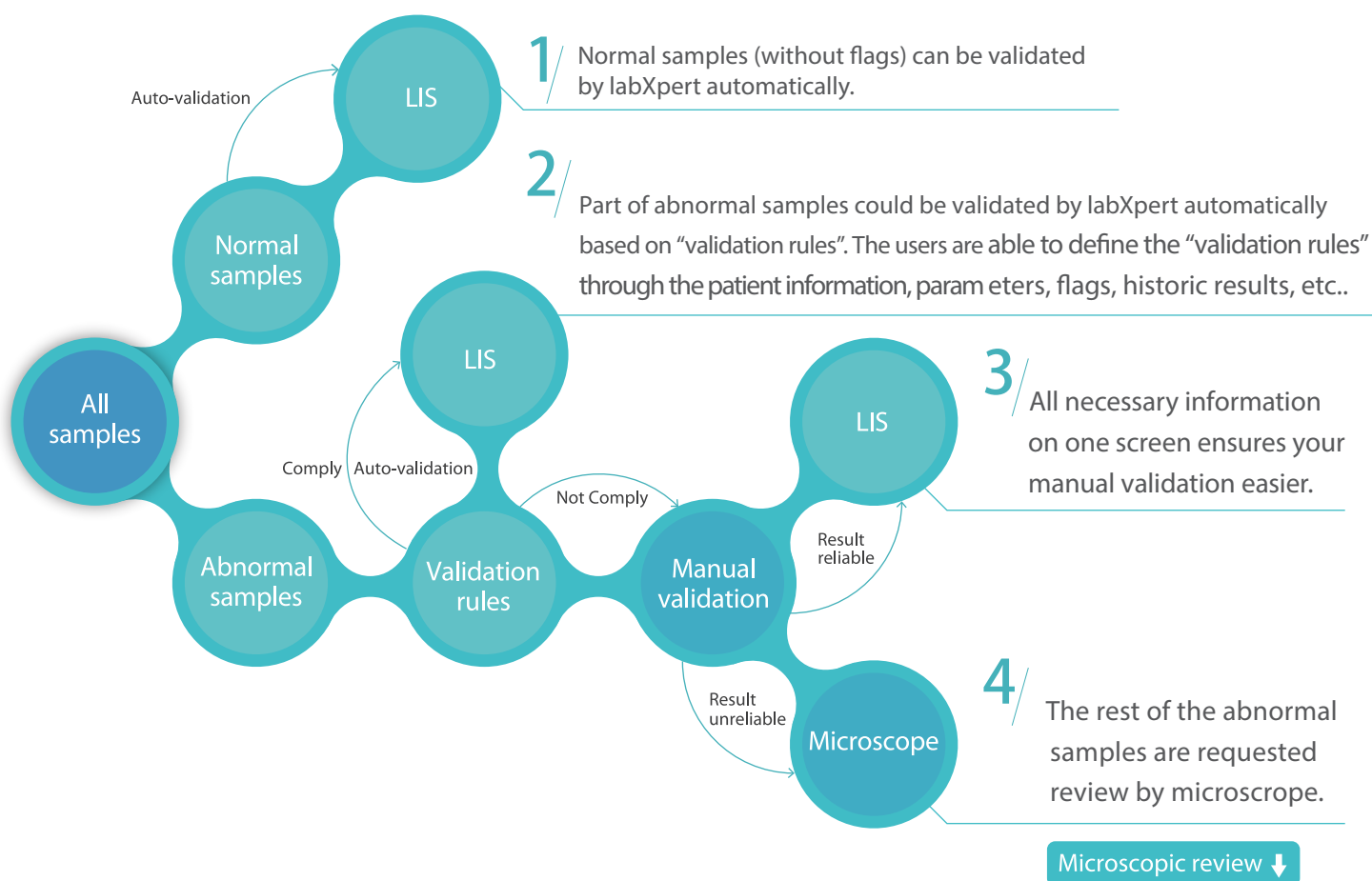

1, 2, 3, 4, work can never be simpler!

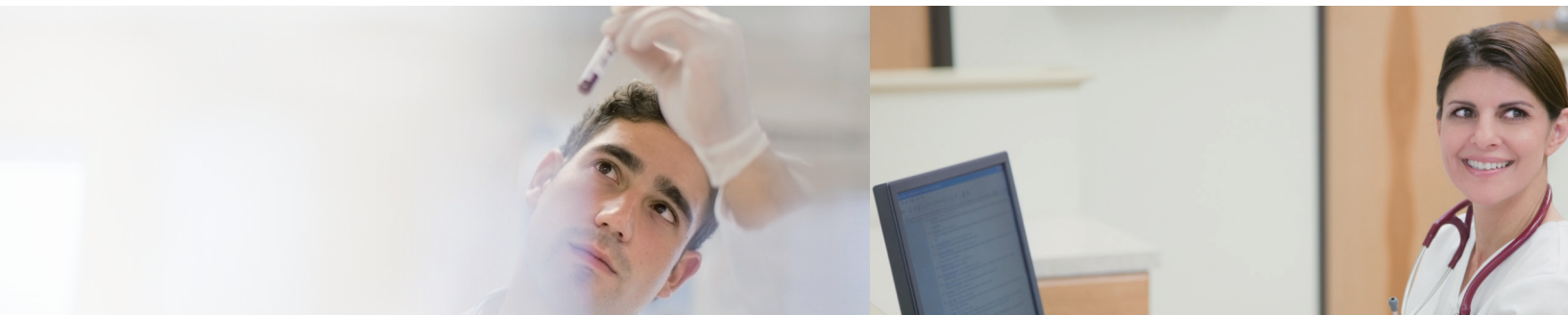

All necessary information on one screen ensures your manual validation easier

Daily workload

Critical value 1st Priority

TAT overtime 2nd Priority

Patient Information

Sample results

Historic results

Re-exam rules

Critical value Message

Graphics

Flags

Expert Tips

Sample Procedure

A more convenient way to input microscopic exam results

Each key in the Numeric Keypad represents one subpopulation of WBC.

The status area shows the number of cells manually categorized, in real time, as well as its targeted total at completion. The WBC DIFF result table area shows the counts and percentages of WBC subpopulations in real time.





Buffer module

Step 1



Samples waiting in the Buffer module


Step 2

Manual review sample results by labXpert

Step 3

Delivering samples for slides, rerun, reflex or stock yard according to orders from labXpert

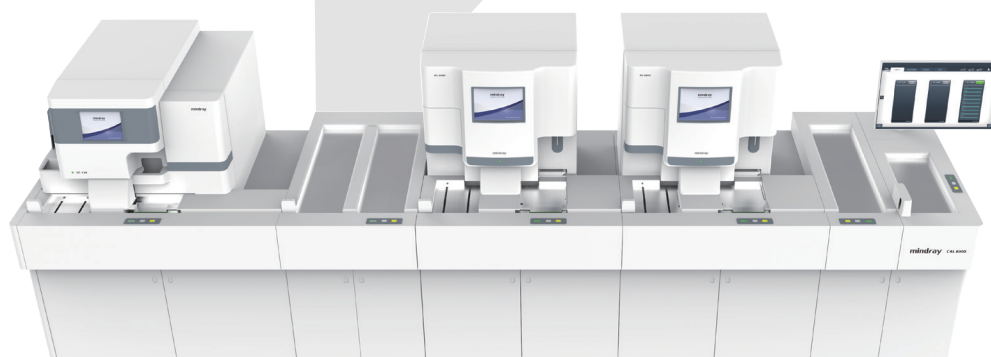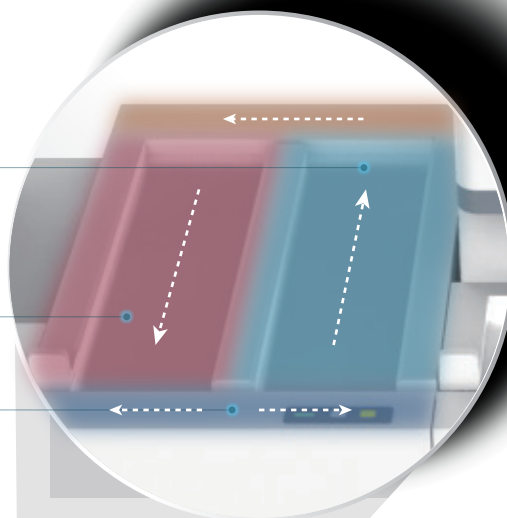

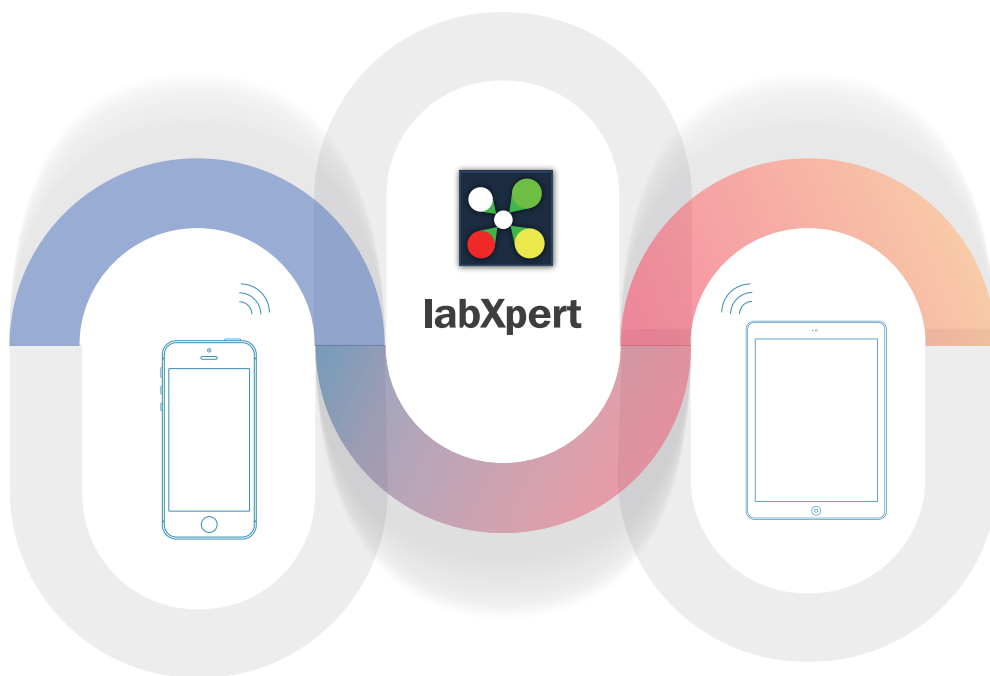

Now you are able to check the sample results through your iPad and iPhone.



Sample results including graphics and statistics of daily sample could be reviewed.





Status of samples

Sample ID	Platform
ST50180005	CAL8000_1
76-5	Unloading Platform
ST50180005	CAL8000_1
76-5	Unloading Platform
ST50180005	CAL8000_1
76-5	Unloading Platform
ST50180005	CAL8000_1
76-5	Unloading Platform
ST50180005	CAL8000_1
76-5	Unloading Platform
ST50180005	CAL8000_1
76-5	Unloading Platform

Review results

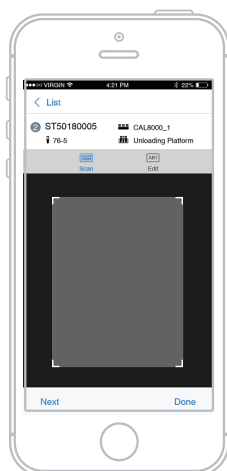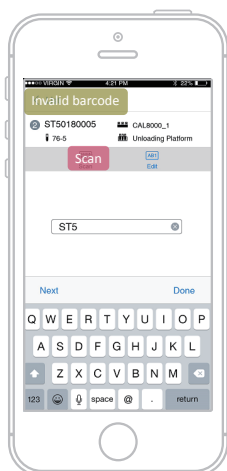

The invalid barcode on the sample tube can be scanned by your iPhone.

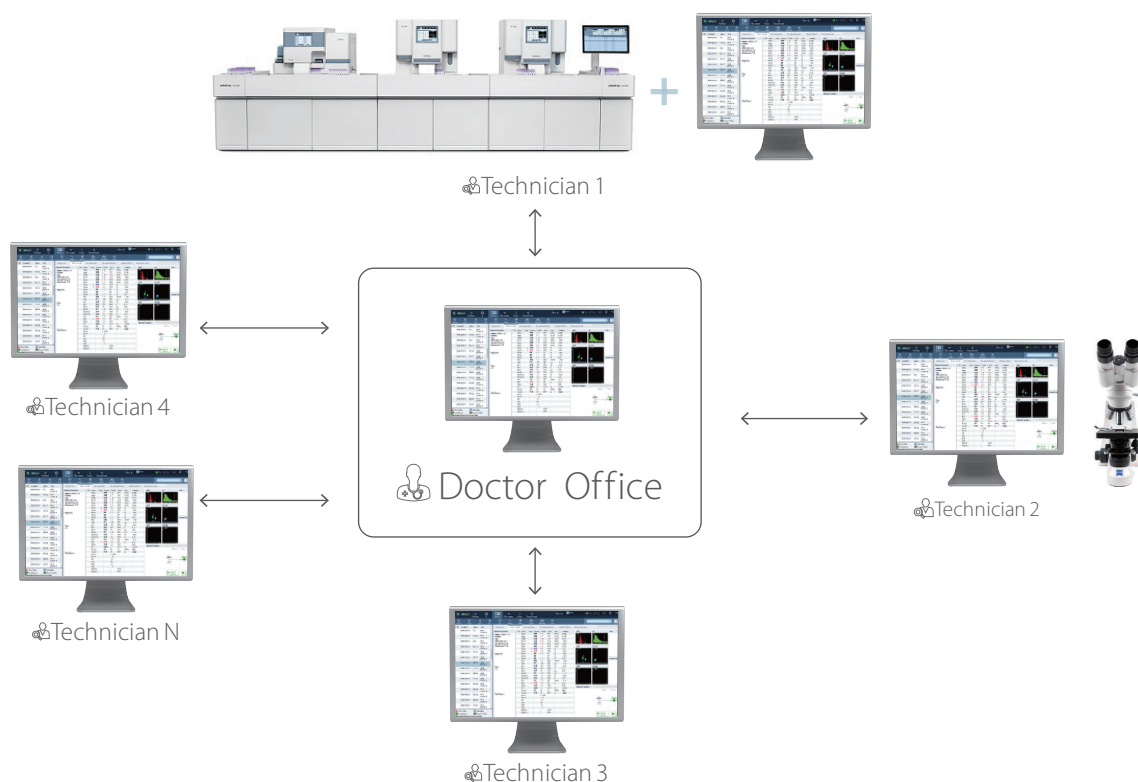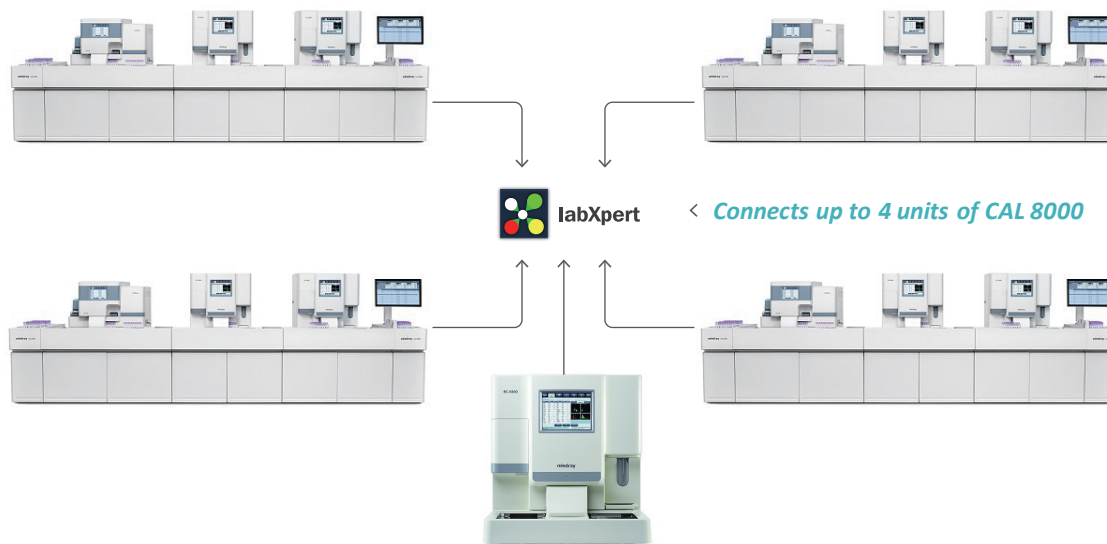

- The labXpert can be installed in many locations.
- One labXpert installed in the doctor office could be used for higher level validation for abnormal samples.
- One labXpert installed next to the CAL 8000 or BC-6800 informs the technicians the status of the validation so that they can take out the samples for further analysis if necessary.
- One labXpert installed next to the microscope simplifies the manual differential work flow.
- All labXperts could communicate with each other.

Mindray Building, Keji 12th Road South,
High-tech Industrial Park, Nanshan, Shenzhen 518057, P.R. China
Tel: +86 755 8188 8998 Fax: +86 755 26582680
E-mail: intl-market@mindray.com www.mindray.com

mindray are registered trademarks or trademarks owned by Shenzhen Mindray Bio-medical Electronics Co., LTD.
© 2015 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved. Specifications subject to changes without prior notice.
P/N:ENG-labXpert-210285x4-20160712

mindray